1. Product Description

bt products Hematology Lyse Reagent is a detergent-based lytic reagent intended to remove RBC 's and platelets from a sample of a whole blood specimen, to permit WBC counting, WBC differential analyzes and hemoglobin concentration determination. It is suitable for in vitro diagnostic use in both automated and semi-automated analyzers.

bt products Hematology Lyse can be used in bt-pro21, bt-pro2400,(+plus),bt-pro2401,(+plus) hematology analyzers.

bt products Hematology Lyse reagent should be used with appropriate **bt products** Brand Hematology Diluent, Hematology Cleaner and other required operational reagents for best results. **bt products** Hematology Lyse reagent is a naturally bacteriostatic and fungi static. All compounds are biodegradable.

2. Reagent Composition

Potassium Cyanide..... < 0.05 %

Quaternary Ammonium Salt. < 3.5 %

pH: 9,0-11,0

Physical form: aqueous solution, limpid.

Precautions: Reagent contains 2-Propanol and Dodecyl trimethylammonium chloride.

H315 Causes skin irritation

H318 Causes serious eye damage

P264: Wash thoroughly after handling

P280: Wear protective gloves/protective clothing/eye protection/face protection.

P310 Immediately call a POISON CENTER or a doctor/physician.

P305+P351+P338 IF IN EYES :Rinse continuously with water for several minutes .Remove contact lenses if present and easy to do.Continue rinsing.

P321 Specific treatment,see on the label

P302+352: IF ON SKIN: Wash with soap and water

P332+313: If skin irritation occurs: Get medical advice/attention

Special precautions: Avoid contact with acid and aqueous acid environment: extremely toxic cyanide acid vapor can be formed.

3. Reagent Storage

Storage conditions: Stored at 8 - 30°C

Product will degrade if exposed to air, keep cap / probe assembly, securely tightened.

Shelf Life: See package label for expiration date.

Discard open containers: 60 days after opening.

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Bilimsel Tıbbi Ürünler Paz.San.Tic.Ltd.Şti.