REF

TUCAL-16  
TUCALL1-08

CONT

2\*8 mL  
1\*8 mL: 2-8 °C  IVD

Please carefully read this package insert before use.

Only use for In Vitro Diagnostic.

**INTENDED USE**

This product is intended for in-vitro use in calibrating the turbidity module of BT URICELL 1600 and URIT UC 1800 series automatic urine analyzers.

**REAGENT**

Turbidity Calibrator is prepared in an aqueous base by adding Forma hydrazine turbidity standards.

**PACKAGE SPECIFICATION AVAILABILITY**

Ref. Code	Size	Value of Turbidity
TUCALL1-08	8mL	600±30NTU

**STORAGE AND STABILITY**

When stored unopened at 2 °C to 8 °C and out of light, This product will be stable for 12 months. Once the calibration solution is opened and stored tightly capped, all calibrator will be stable for 30 days at 2 °C to 8 °C. This product should never be frozen.

**PROCEDURE**

1. Read the product manual carefully before using, performs on calibration in strict accordance with the specifications.
2. Before testing, make operating temperature to reach 18 - 30 °C and relative humidity no more than 80%.
3. Take out the calibration solution and return to room temperature, after blending upside down 3-5 times, pour the calibration solution into a dry, clean test tube, perform on calibration in strict accordance with the specifications of the automatic urine analyzer. The Calibration should be finished in 10 minutes, otherwise blending again.

**PRODUCT PERFORMANCE**

Marked value of Turbidity = 600NTU. The accuracy of assignment: Turbidity=600±30NTU

**CAUTIONS**

1. This product is for professional use only.
2. Ensure the test tube of clean, dry. Residual detergents or other substances may influence calibration.
3. In order to ensure the calibration results are accurate and reliable, complete calibration as soon as possible after blending the calibration solution.
4. If the liquid touches the eyes and skin, rinse with water immediately.
5. Calibration solution should be saved in the required storage conditions.

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Bilimsel Tıbbi Ürünler Paz.San.Tic.Ltd.Şti.