Please carefully read this package insert before use.

Only use for In Vitro Diagnostic.

INTENDED USE

Urinalysis control is intended for urine analyzer and matched urine reagent strips in order to monitor the quality of urinalysis test.

It is used as strip control in automatic and semi-automatic urine chemistry devices. Suitable for BT Uricell 1600, URIT UC 1800 and BT Uri 500B& BTUri 50 analyzers.

SUMMARY

Urinalysis control is a ready-to-use liquid preparation that works with urine analyzer and matched urine reagent strips.

NO.I is Negative control, NO.II is Positive control, NO.III is Positive control for Ascorbic Acid only.

TYPE	QUALITY CONTROL TEST INSERT
UQ11-08 11 pm. Control	pH, Nitrite, Glucose, Specific Gravity, Blood, Protein, Bilirubin, Urobilinogen, Ketone, Leukocytes(WBC), Ascorbic Acid
UQ14-08 14 pm. Control	pH, Nitrite, Glucose, Specific Gravity, Blood, Protein, Bilirubin, Urobilinogen, Ketone, Leukocytes(WBC), Ascorbic Acid, Creatinine, Calcium, Micro Albumin

REAGENT

Urinalysis Control is prepared in aqueous base by adding chemicals, constituents of animal origin, preservatives and stabilizers. This product does not contain the substances extracted from urine.

PROCEDURE

This product should be treated the same as patient specimen and run according to the instructions of the instrument, kit, or reagent.

1. Before testing, make operating temperature to reach 18 -30 °C .
2. Unscrew the cap and draw the liquid in the bottle across all reaction areas of the urine reagent strip, thoroughly saturating each pad. Or you can also pour out the liquid into the test tube and dip the urine reagent strip into it. When using with the test tube, it could supply 6 times tests per tube at most. Don't mix the used with the unused.

LIMITATIONS

1. This product is not intended for use as a standard.
2. If there is evidence of microbial contamination or excessive turbidity in the product discard the vial.
3. This product should not be used past the expiration date.

CAUTIONS

1. Please replace cap tightly and store it at 2-8 °C after each use.
2. If the product is opened too long or the environmental temperature is not at 18 -30 °C , the testing result maybe low or high.
3. This product should be treated the same as patient specimens. After use, dispose as the medical waste. If the liquid touches the eyes and skin, clean it immediately.

STORAGE AND STABILITY

This product will be stable until the expiration date when stored unopened at 2-8 °C and out of light. Once the control is opened and stored tightly capped, all analytes will be stable for 15 days at 2-8 °C . This product should never be frozen.

ASSIGNMENT OF VALUES

Enclosed quality control results are only the reference of this certain batch. Variations over time and between laboratories may be caused by Differences in laboratory technique, instrumentation and reagents or by manufacture test method modifications. Each laboratory should use the results provided only as a reference and establish its own parameters of precision.

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Bilimsel Tıbbi Ürünler Paz.San.Tic.Ltd.Şti.