REF

CONT

URF1-221000

1 x 1000 mL

: 2-35 °C

**INTENDED USE**

Uricell 1600 **MAINTENANCE REAGENT** is used for rinsing of flow system of Urine sediment analyzer at maintenance and repairing. This product is used for BT URICELL 1600 and URIT UC 1800 automatic urine analyzers.

**FUNCTIONS**

With the functions of emulsification and catalysis, the detergent can emulsify the fat, distribute the solidified dirt and transform the protein into a water solubility amino acid to maintain the cleanness of pipelines and flow cell. With the good moist function, it also can discharge bubbles does not impact of the instrument's use life, and will not colorate and corrupt the pipelines.

**REAGENT COMPOSITION**

Sodium chloride, sodium azide and detergent

**REAGENT PREPARATION**

Ready to use

**STABILITY AND STORAGE**

1. Store in dark place, keep away from direct sunlight, temperature should be 2 - 35°C.
2. Expire date 24 month that sealed storage and avoid exposing to direct sunlight; 90 days after unscrewing the detergent container.

**OPERATIONAL PROCEDURE**

Details please refer to the operation manual of the urine sediment analyzer.

**SPESIFICATIONS**

1. The pH value is 7,0-8,0 (25 °C ± 1 °C)
2. Blank value: when you do the blank test in the urine sediment analyzer, the total number of particles value should be 0.

**RESTRICTION, PRECAUTION AND NOTICE**

1. Use the detergent under the professional technician's guidance.
2. Make sure that the using reagent is suitable for your analyzer any mismatch will lead to misleading results.
3. Avoid contact with skin and eyes. In case of contact, rinse immediately with plenty of water or seek medical advice.
4. Avoid inhaling the reagent.
5. Tight up the container's cap to prevent from volatilization and contamination.
6. Dispose the remains after 60 days of use after unscrewing the detergent container.
7. If product has been frozen, let it warm to room temperature, and gently invert product to mix it completely. Verify background results before analyzing patient samples.

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Bilimsel Tıbbi Ürünler Paz.San.Tic.Ltd.Şti.