REF

URF2-115000

CONT

1 x 5000 mL

: 2-35 °C  IVD**INTENDED USE**

URICELL 1280 REAGENT A is suitable for daily maintenance and rinsing the flow system, as well as sample dilution.

**FUNCTIONS**

With the functions of emulsification and catalysis, the detergent can emulsify the fat, distribute the solidified dirt and transform the protein into a water solubility amino acid to maintain the cleanness of pipelines and flow cell. With the good moist function, it also can discharge bubbles does not impact of the instrument's use life, and will not colorate and corrupt the pipelines.

**REAGENT COMPOSITION**

Sodium chloride, buffer and surfactant.

**REAGENT PREPARATION**

Ready to use

**STABILITY AND STORAGE**

1. Store in dark place, keep away from direct sunlight, temperature should be 2 - 35°C.
2. Expire date 24 month that sealed storage and avoid exposing to direct sunlight; 90 days after unscrewing the detergent container.

**OPERATIONAL PROCEDURE**

Details please refer to the operation manual of the urine sediment analyzer.

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1. The pH value is 7,6±0,5 ( 25 °C ± 1 °C )
2. Blank value: when you do the blank test in the urine sediment analyzer, the total number of particles value should be 0.

**RESTRICTION, PRECAUTION AND NOTICE**

1. Use the detergent under the professional technician's guidance.
2. Make sure that the using reagent is suitable for your analyzer any mismatch will lead to misleading results.
3. Avoid contact with skin and eyes. In case of contact, rinse immediately with plenty of water or seek medical advice.
4. Avoid inhaling the reagent.
5. Tight up the container's cap to prevent from volatilization and contamination.
6. Dispose the remains after 60 days of use after unscrewing the detergent container.
7. If product has been frozen, let it warm to room temperature, and gently invert product to mix it completely. Verify background results before analyzing patient samples.

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Bilimsel Tıbbi Ürünler Paz.San.Tic.Ltd.Şti.