URF2-14100

1 x 100 mL

: 2-35 °C

**INTENDED USE**

URICELL 1280 REAGENT C is used for rinsing of Urine sediment analyzer's counting chamber.

**FUNCTIONS**

With the functions of emulsification and catalysis, the detergent can emulsify the fat, distribute the solidified dirt and transform the protein into a water solubility amino acid to maintain the cleanness of pipelines and flow cell. With the good moist function, it also can discharge bubbles does not impact of the instrument's use life, and will not colorate and corrupt the pipelines.

**REAGENT COMPOSITION**

Sodium hypochlorite .

**REAGENT PREPARATION**

Ready to use

**STABILITY AND STORAGE**

1. Store in dark place, keep away from direct sunlight, temperature should be 2 - 35°C.
2. Expire date 24 month that sealed storage and avoid exposing to direct sunlight; 90 days after unscrewing the detergent container.

**OPERATIONAL PROCEDURE**

Details please refer to the operation manual of the urine sediment analyzer.

**SPEŞIFICATIONS**

1. The pH value is 11,0-13,0 ( 25 °C ± 1 °C )
2. Blank value: when you do the blank test in the urine sediment analyzer, the total number of particles value should be 0.

**RESTRICTION, PRECAUTION AND NOTICE**

1. Use the detergent under the professional technician's guidance.
2. Make sure that the using reagent is suitable for your analyzer any mismatch will lead to misleading results.
3. Dispose the remains after 60 days of use after unscrewing the detergent container.
4. If product has been frozen, let it warm to room temperature, and gently invert product to mix it completely. Verify background results before analyzing patient samples.

Product PH > 11: Skin Corrosive 1B H314 Causes severe skin burns and serious eye damage

P260: Do not breathe dust/fume/gas/mist/vapours/spray.

P264: Wash thoroughly after handling

P280: Wear protective gloves/protective clothing/eye protection/face protection.

P301 + P330 + P331: IF SWALLOWED: rinse mouth. Do NOT induce vomiting.

P303 + P361 + P353: IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower

P304 + P340: IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing

P305 + P351 + P338: IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing

P363: Wash contaminated clothing before reuse.

P310: Immediately call a POISON CENTER or doctor/physician

P321: Specific treatment (see ... on this label).

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Bilimsel Tıbbi Ürünler Paz. San. Tic. Ltd. Şti.